



Stability and comfort for noninvasive ventilation



Light and unobtrusive, Quattro™ FX Non-vented (NV) Full Face Mask provides reliability and comfort for noninvasive ventilation in the hospital or the home.

Benefits for you ...

The comfort, performance and feeling of openness that Quattro FX NV provides, help patients accept and continue therapy in the hospital or the home.

The mask gives you the tools to achieve the best fit for your patients. The custom-made sizing tool and **Swing Seal** fitting technique simplify the sizing, fitting and sealing process, so fitting Quattro FX NV is fast and straightforward.

Quattro FX NV—fast and straightforward to fit.

... and your patients

Because Quattro FX NV is a full face mask that **starts mid-way on the nasal bridge**, it touches the patient's face only where necessary. This not only reduces the chances of skin irritation, but creates a clear field of vision and less obstruction for the patient.

With new technologies in the Spring Air cushion and Spring frame, Quattro FX NV offers comfort and a secure seal in a less obtrusive full face mask.

Quattro FX NV—comfort and a secure seal in a smaller full face mask.



Quattro™ FX NV
NON-VENTED FULL FACE MASK

Technology highlight

Size. Swing. Seal **NEW**

Sizing, fitting and sealing Quattro FX NV is fast and straightforward.



The custom-made **sizing tool** provides an easy and accurate sizing process.

Once sized correctly, fitting the mask is easy too, with the **Swing Seal** technique:



1 Position the cushion at an angle on the nasal bridge ...



2 ... then **swing** the lower half of the mask down against the face to **seal**.

Clear line of sight

and feeling of openness created by a full face mask that starts mid-way on the nasal bridge

Clear (non-tinted) frame

Clear view enables the caregiver to more easily monitor the patient's face for changes during therapy

Flexible, durable Spring frame

strong enough to stabilise the mask on the face, soft enough to accommodate user movement

360° elbow rotation

positions the tubing for ease of use

Blue-tinted elbow for added safety

Specifically designed so that it cannot be interchanged with the clear elbow of ResMed's vented masks

New, tension-reducing headgear design

features a contoured crown strap that sits snug at the top of the head and avoids the sensitive area around the back of the neck



Contoured cushion membrane around sensitive nasal bridge area rolls and inflates to gently meet the nose rather than stretch over it

Dual-wall, Spring Air cushion

provides soft, even pressure distribution

Set-and-forget headgear clips

easy to grip for attaching and detaching; retain desired headgear settings

Product Codes

	Asia Pacific
Small	61755
Medium	61756
Large	61757

Quattro™ FX NV
NON-VENTED FULL FACE MASK